HEINE QUALITY
MADE IN GERMANY

⚡ **HEINE® LoupeLight 2**
HEINE® MicroLight 2

HEINE Optotechnik GmbH & Co. KG
Dornierstr. 6 · 82205 Gilching · Germany
E-Mail: info@heine.com · www.heine.com

MED 113729 2020-06-02

V-200.00.039

HEINE® LoupeLight 2

HEINE® MicroLight 2

DEUTSCH	3
ENGLISH	7
FRANÇAIS	11
ESPAÑOL	15
ITALIANO	19
SVENSKA	23
NEDERLANDS	27
DANSK	31
NORSK	35
SUOMI	39
PORTUGUÊS	43

HEINE® LoupeLight 2

HEINE® MicroLight 2

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HEINE® LoupeLight 2

HEINE® MicroLight 2

Please read and follow these instructions for use of and keep them for future reference. (Background: blue, foreground: white.)

Intended Use

HEINE headlights are battery-operated devices intended to be worn on the users head to provide a secondary light source to aid visualization during surgical, diagnostic, or therapeutic procedures. It should only be operated by trained medical staff within a professional health facility.

For U.S. only:

Federal law restricts this device to sale by or on the order of a Physician or Practitioner.

Warnings and Safety Information

CAUTION! Indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations of mild to moderate extent. (Background color yellow; foreground color black).

NOTE! Note indicates valuable advice in terms of installation, operation, maintenance or repair. Notes are important, but not related to hazardous situations.

Setting up

Only operate the **LoupeLight 2** and the **MicroLight 2** with the HEINE® mPack mini. Ensure safe cable routing (see Figs. 1 and 2). Connect the cable to the lamp housing by firmly tightening the coupling nut on the cable end to the counterpart on the lamp housing. Exposure to high mechanical strains and kinking of electrical cables on the headlights must be prevented.

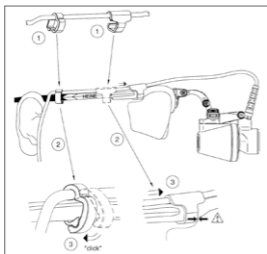
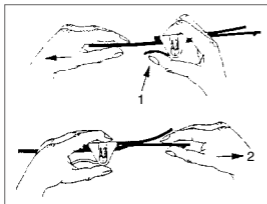
Fig. 1 Safe cable routing S-Frame

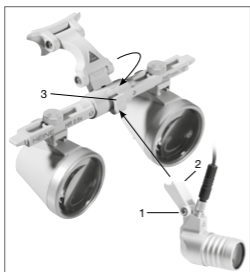
Fig. 2 Safe cable routing headband Lightweight

Adjustment of the **headband Lightweight** (see Fig. 3): To adjust the head size, the headband can be widened by pressing the lever (1) and tightened by pulling at the free end (2).

Fig. 3 Headband Lightweight

How to fit the LoupeLight 2 to the binocular loupe mount with i-View® (Fig. 4):
 Unscrew the screw (1) on the light mount (2) so that this can be attached to the round attachment of the loupe mount (3). The LoupeLight 2 housing has to fit between the eyepieces (not above them!).

The cable of the **LoupeLight 2** can be affixed on the right or left side piece of spectacles using the cable clips. The enclosed coloured retention clips are designed for personalisation and labelling of the loupe light.

Fig. 4 Fitting the LoupeLight 2 with i-View®

Operation

Use the connecting cord cable to connect the LED light to the mPack mini (note the shape of the plug!).

Use the knob of the mPack mini to switch the LED light on and adjust the head of the light vertically such that the examination field is illuminated centrally.

● Jerky, fast movements can alter the position of the lamp housing.

Tuning the swivel lever at the mPack mini allows continuous adjustment of the brightness.

Fasten the cable on your clothes using the support clip provided (see Fig. 5).


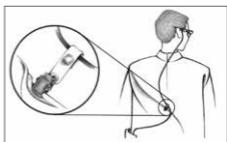
Fig. 5 Cable fastening with support clip

LoupeLight 2

Adjust the binocular loupe for the intended application. Be sure the eyepieces are positioned as symmetrically as possible on the optics holder (take off the loupe and check this), as otherwise the spot of light will appear shifted to the side.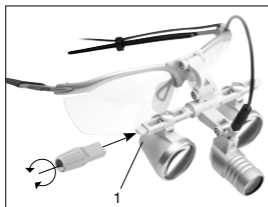

LoupeLight 2 on loupe mount with i-View®.

If required, a steri-lever (see Fig. 6) can be attached to one end (1) of the loupe mount so that the binocular loupe can be swivelled independently of the light (only HR/HRP Loupes).

Fig. 6 Using the steri-lever

MicroLight 2

In order that the MicroLight 2 remains in place in every gradually and freely selectable position, it may be necessary to retighten the screws using a screwdriver (see Fig. 7).

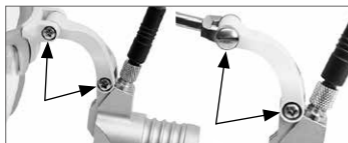
Fig. 7 Tighten the screws

 Danger from loose, falling parts. Please check before every use that all the parts are tightened safely.

Do not shine the light directly into the eyes.

Never ever hold magnifying glasses or any other optical system in front of the light of the LED-headlight.

Consumable material and options

For applications in the dental area:

If necessary, use a yellow filter in order to avoid the composite filling material hardening, as this blocks UV radiation.

The plug-on polarisation filter P2 enables a differentiation and low-reflex examination of tissue surfaces due to polarized light. (Only in combination with LoupeLight 2 and HR 2.5x binocular loupes, ready to use.) The polarisation filter P2 is not a replacement for a yellow filter as it may harden composite filling material.

The setup and operation of the HEINE power sources are described in a separate instruction document.

Hygienic reprocessing

The instruction is available:

- online as a PDF at www.heine.com

- in a paper version which you can request from the address listed

Maintenance

Cable switching

Switch the mPack mini off by turning the controller to the zero position. Open and/or remove all cable holders such that the cable is exposed. Disconnect the plug on the mPack mini and unlock the plug on the light. Connect the new cable to the light and the mPack mini. Attach the cable to the light.

We as manufacturer do not stipulate a safety inspection in accordance with § 6 Medical Device Operator Ordinance (MPBetreibV), ref. attachment 1, for the HEINE product.

Service

The device has no components serviceable by the end-user.

General Notes

 The warranty for the entire product is invalidated if non-genuine HEINE products or non-original parts are used and if repairs or modifications are made to the device by persons not authorized by HEINE. For more information, please visit www.heine.com.

The expected life cycle, when the device is normal used and the warning and safety information as well as the maintenance instructions are observed, is up to 7 years. Beyond this period, the product may continue to be used if it is in a safe and good condition.

Store and use the device in dry and dust-free environments only!

You may want to have a connecting cord ready for use.

General Warnings

 Check the correct operation of the device before use! Do not use the device if there are visible signs of damage.

Do not use the device in fire- or explosive risk area (e.g. oxygen saturated or anesthetic environments).

Do not modify the device.

The device is not sterile. Avoid touching with sterile hands, areas or instruments.

Only use the lens protector as transport protection. Store the lens protector in the transport case. Do not use in the surgical rooms.

Note the operating time of the power source. You may want to have a charged spare power source ready for use.

Disposal

The product must be recycled as separated electrical and electronic devices. Please observe the relevant state-specific disposal regulations.

Electromagnetic Compatibility

Medical electric devices are subject to special precautionary measures with regard to electromagnetic compatibility (EMC). Portable and mobile high frequency communication equipment can affect medical electric devices.

The appendix contains the tables „Guidance and Manufacturer's declaration – Electromagnetic immunity“, technical specification and an explanation of the used symbols.

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Electromagnetic disturbances – Requirements and tests

The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such environments.

Statement for the operational environments	Inside hospitals except for: near active HF surgical equipment and the RF shielded room of an ME system for magnetic resonance imaging, where the intensity of EM disturbances are high.
Performance features of the ME system that have been determined to be essential to the performance	None
Warning	Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.
	Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.
	Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the Headlights, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.
A list of all cables, transducers and other accessories that are relevant for the EMC compliance	See attachment
Test	Compliance
RF emissions CISPR11	Group 1 Class B
Harmonic Emissions	Not applicable
Voltage Fluctuations/Flicker	Not applicable

Technical specification

Nominal voltage	3 V DC
Nominal current	max. 350 mA
Protection class	Internally powered (accu)
Illuminance (without filters)	typ. 55,000 lx
Colour temperature	typ. 4,000 K
Colour rendering index (CRI)	typ. ≥ 90
Cable length	approx. 1.50 m
Light field diameter at a 420 mm working distance	approx. 80 mm
Vertical direction of illumination	- 90° to $\geq 90^\circ$ - 20° to $\geq 20^\circ$
Weight	approx. 50 g (only light) approx. 80 g (with S-Frame) approx. 190 g (with Lightweight Headband)
Operation time	typ. 4 h (with mPack mini continuous operation)
Environmental conditions for operation	+10 °C to +35 °C 30 % to 75 % rel. humidity 700 hPa to 1060 hPa
Environmental conditions for storage	+5 °C to +45 °C 45 % to 80 % rel. humidity 500 hPa to 1060 hPa
Environmental conditions for transport	-20 °C to +50 °C 45 % to 80 % rel. humidity 500 hPa to 1060 hPa
Device classification according to IEC 62471	Group 2

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Explanation of utilized symbols

The following symbols are used on the device or on the packaging:

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