
HEINE QUALITY
MADE IN GERMANY

HEINE® ML4 LED HeadLight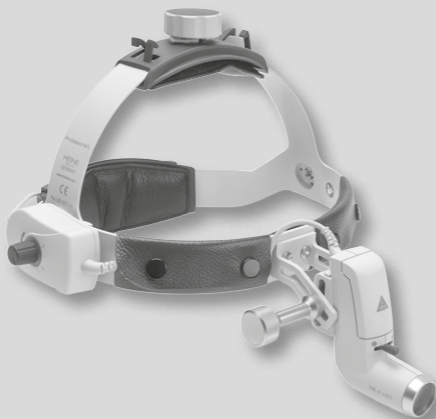

HEINE Optotechnik GmbH & Co. KG
Dornierstr. 6 · 82205 Gilching · Germany
E-Mail: info@heine.com · www.heine.com

MED 113192 2020-06-02

V-200.00.019

HEINE® ML4 LED HeadLight

DEUTSCH	3
ENGLISH	7
FRANÇAIS	11
ESPAÑOL	15
ITALIANO	19
SVENSKA	23
NEDERLANDS	27
DANSK	31
NORSK	35
SUOMI	39
PORTUGUÊS	43

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HEINE® ML4 LED HeadLight

Please read and follow these instructions for use and keep them for future reference.

Intended Use

Supplied internally with electrical power, the headlight is designed for short-term local illumination of the body, with the exception of the eyes, to support diagnosis during a therapeutic or surgical intervention.

The instrument may be used only by qualified medical personnel in a professional health facility.

For U.S. only:

 Federal law restricts this device to sale by or on the order of a Physician or Practitioner.

Warnings and Safety Information

 **CAUTION!** This symbol indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations of mild to moderate extent. (Background color yellow; foreground color black).

 **NOTE!** This symbol indicates valuable advice in terms of set up, operation, maintenance or repair, as applicable. Notes are important, but not related to hazardous situations.

Setting up

Only operate the **ML4 LED** with the HEINE® mPack, the mPack UNPLUGGED or the plug transformer for the headband rheostat (Cat.No. X-095.16.330).

Take care that the cable is fastened tightly to the headband in the provided buckles (see Fig. 1).

Exposure to high mechanical strains and kinking of electrical cables on the headlights must be prevented.

Fig. 1 Safe cable routing headband ML4 LED

Adjustment of the **headband Professional L** (see Fig. 2): To adjust for the head size, the headband can be widened or tightened by turning the turnbuckles at the top (1) and the back. (2).

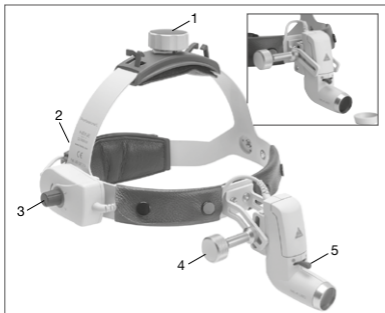
Fig. 2 Headband Professional L (see tall fig.)
optical protection cover (see small fig.)

Fitting the headband rheostat

You can fit the headband rheostat (1) either on the left or the right side of the headband (see Fig. 3). The headband rheostat (1) can then be used for stepless adjustment of the brightness.

It is recommended that the mPack rheostat is set to „0“.

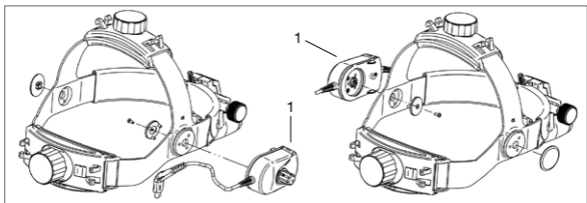
Abb. 3 Fitting the headband rheostat

Operation

To set up the ML4 LED with mPack or EN50, connect the short cord from the brightness control with the connecting cord mPack UNPLUGGED/Cinch and fix this to the back part of the headband (Fig. 4). Connect the instrument to the Cinch-socket of the mPack or EN 50. The rheostat (1) (Fig. 3) is used to adjust the brightness and to turn the device on and off.

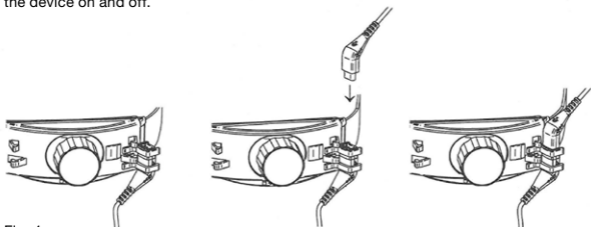
Fig. 4

To use the ML4 LED with the mPack UNPLUGGED first remove the rear cover of the headband (Fig. 5).

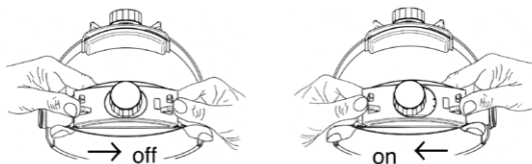
Fig. 5

Then attach the mPack UNPLUGGED to the rear of the headband. To do so, hold the headband in your left hand and push the mPack UNPLUGGED to the left, until it clicks into place (Fig. 6).

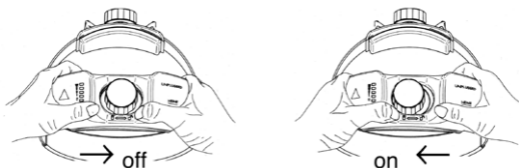
Fig. 6

Selecting viewing angle and field of view (Fig. 2)

Without S-Guard

After loosening the locking screw (4), the light head can be inclined and the angle of the light direction can be adjusted.

After adjustment, re-tighten the screw (4).

With S-Guard

Before putting on the loupes, select your inter-pupillary distance (PD) or an average value on the scale. Unscrew the retaining screw and slide the eyepieces to one side symmetrically.

Unscrew the fixing screw (4) and locate the nosepad on the bridge of the nose.

Adjust the eyepieces so that they nearly touch the protective lenses.

Shine the ML4 LED onto an object at your normal working distance and adjust the eyepieces so that the object is centered in both eyepieces. After adjustment, re-tighten the screw (4). The loupes are correctly adjusted when you see a clearly-defined, round, shadow-free field of view. The slider (5) allows up or down adjustment of size of the light field.

Consumable material and options

For applications in the dental area:

If necessary, use a yellow filter in order to avoid the composite filling material hardening, as this blocks UV radiation.

The plug-on polarisation filter P2 enables a differentiation and low-reflex examination of tissue surfaces due to polarized light. Only in combination with HR 2.5x binocular loupes, ready to use. The polarisation filter P2 is not a replacement for a yellow filter as it may harden composite filling material.

The setup and operation of the HEINE power sources are described in a separate instruction document.

Hygienic reprocessing

The instruction is available:

- at www.heine.com
- in a paper version which you can request from the address listed

Maintenance

The device is maintenance free.

Service

The device has no components serviceable by the end-user.

General notes

 The warranty for the entire product is invalidated if non-genuine HEINE products or non-original parts are used and if repairs or modifications are made to the device by persons not authorized by HEINE. For more information, please visit www.heine.com.

The expected life cycle amounts with designated use and the observation of warning and safety information as well as the maintenance instructions up to 7 years. Beyond this period, the product may continue to be used if it is in a safe and good condition.

Store and use the device in dry and dust-free environments only!

General Warnings

 Check the correct operation of the device before use! Do not use the device if there are visible signs of damage.

Do not use the device in fire- or explosive risk area (e.g. oxygen saturated or anesthetic environments)

Do not modify the device.

Use only original HEINE parts, spare parts, accessories and power sources.

The device and its accessories are not sterile. Avoid touching with sterile hands, areas or instruments.

Danger from loose, falling parts. Please check before every use that all the parts are tightened safely.

Route the supply cable in such a way that nobody can trip over it.

Do not shine the light directly into the eyes.

Never ever hold magnifying glasses or any other optical system in front of the light of the LED-headlight.

Only use the lens protector as transport protection. Store the lens protector in the transport case. Do not use in the surgical rooms.

Note the operating time of the power source. You may want to have a charged spare power source ready for use.

Jerky, fast movements can alter the position of the lamp housing.

Disposal

 The product must be recycled as separated electrical and electronic devices. Please observe the relevant state-specific disposal regulations.

The appendix contains following tables

- Electromagnetic disturbances – Requirements and tests
- Technical specification
- Explanation of the used symbols

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Electromagnetic disturbances – Requirements and tests

The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such environments.

Statement for the operational environments	Inside hospitals except for: near active HF surgical equipment and the RF shielded room of an ME system for magnetic resonance imaging, where the intensity of EM disturbances are high.
Performance features of the ME system that have been determined to be essential to the performance	None
Warning	Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.
	Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.
	Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the ML4 HeadLight, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.
A list of all cables, transducers and other accessories that are relevant for the EMC compliance	As applicable, see attachment
Test	Compliance
RF emissions CISPR11	Group 1* Class B
Harmonic Emissions	Passed*
Voltage Fluctuations/Flicker	Passed*

*n/a: "Not applicable" in the internally powered mode

Technical specification


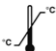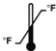
Nominal voltage	6-9 V DC
Nominal current	max. 600 mA
Protection class	II (power supply) Internally powered (accu)
Illuminance (without filters)	typ. 90,000 lx at 180 mm distance
Colour temperature	typ. 5,500 K
Colour rendering index (CRI)	typ. 92, min. 90
Cable length	approx. 1.60 m
Light field diameter at a 420 mm working distance	30-80 mm
Weight	approx. 70 g (light head rheostat) approx. 260 g (with Headband Professional L) approx. 460 g (with mPack UNPLUGGED)
Operation time	typ. 8.5 h (with mPack) typ. 3.5 h (with mPack UNPLUGGED)
Environmental conditions for operation	+10 °C to +35 °C 30 % to 75 % rel. humidity 700 hPa to 1060 hPa
Environmental conditions for storage	+5 °C to +45 °C 45 % to 80 % rel. humidity 500 hPa to 1060 hPa
Environmental conditions for transport	-20 °C to +50 °C 45 % to 80 % rel. humidity 500 hPa to 1060 hPa
Device classification according to IEC 62471	Group 2

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